

# SAFETY BLOOD COLLECTION SET



VacuPro Safety 



DESCRIPTION	QTY/BX
SAFETY BLOOD COLLECTION SET With Luer MS Adapter, 21 gauge 3/4 in, 12 in Tube.	100
SAFETY BLOOD COLLECTION SET With Luer MS Adapter, 23 gauge 3/4 in, 12 in Tube.	100
SAFETY BLOOD COLLECTION SET With Luer MS Adapter, 25 gauge 3/4 in, 12 in Tube.	100

  
APPROVED

# SAFETY BLOOD COLLECTION SET

## ONE-HANDED TECHNIQUE

Upon completion of collection, apply light pressure to site using three fingers as shown.



Withdraw safety blood collection set by holding the yellow safety shield grip area with the thumb and index finger.



Then while holding the tubing securely with the other 3 fingers



push the yellow safety shield forward with thumb and index finger



until the needle is completely covered and a click is heard, indicating that the safety shield is locked in place over the needle tip.



Discard immediately into an approved sharps container.

## TWO-HANDED TECHNIQUE

Upon completion of collection, apply light pressure to site using three fingers as shown.



Withdraw safety blood collection set by holding the yellow safety shield grip area with the thumb and index finger.



With the opposite hand, hold the tubing between thumb and index finger.



Push the yellow shield forward until the needle is completely covered and a click is heard, indicating that the safety shield is locked in place over the needle tip.



Discard immediately into an approved sharps container.