

Safety Blood-Collecting Needles

✓ Instruction for use



Try to make upper white protect cap face upwards with one hand and take bottom protect cap with the other hand. Rotate white protect cap and keep parallel to the rubber cap. Leave rubber cap still on the needle cannula when we remove the white protect cap.



Take bottom protect cap with one hand and take needle holder with the other hand. Put clockwise rotation of blood collection needle into the holder and lock the needle completely.



Open safety device of the blood collection needle until it is locked. Then remove the bottom protect cap of needle tip.



Vein puncture according to the vein infusion method of clinical nursing. Put the other end of blood collection needle into vacuum tube of blood sample collection. Blood will flow automatically into blood collection tube under differential pressure.



After collection of the blood, holding finger press down on the safety device of the holder with two hands. Make safety device lock the needle completely.




After collection of the blood, press the safety device, lever side down, onto a flat surface to cover the needle and lock the safety device, i.e., bedside table. We are free to use either of above two methods.



✓ Product features:

1. safety device+needle holder
Avoid needlepoint stabbing and reduce blood infection during clinical application.

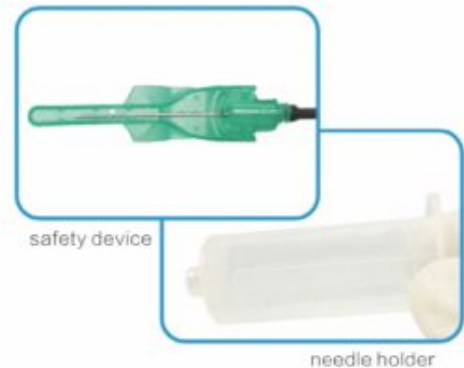








2. safety lock in safety device
There is a specific design of safety lock in safety device, and it's more preferable for nurses to operate.



✓ Intended use

This kind of product which is used for blood collection, is applicable for needle holder and vacuum tube for blood sample collection.

Gauge	OD(mm)	Color
25G	0.50	orange 
24G	0.55	mid-purple 
23G	0.60	deep blue 
22G	0.70	black 
21G	0.80	deep green 
20G	0.90	yellow 
19G	1.10	milk 
18G	1.20	pink 